



The **ViVo** Mouse®

Versions: Lite, Standard & Professional

Quick Start Guide

Copyright © 2010-2014 Vortant Technologies, LLC

Table of Contents

ViVo Mouse Quick Start Guide	3
Getting Started	3
Step 1. Install the Web Cam	4
Step 2. Install the ViVo Mouse	5
Step 3. Run the ViVo Mouse	6

ViVo Mouse Quick Start Guide

Getting Started

Thank you for purchasing the ViVo Mouse! We hope you have many enjoyable experiences using the product with your computer. This guide should help you get started. For full details, please see the ***Installation Guide*** and the ***User's Guide***.

Technical Support

Fax: (828) 237-4141


E-mail: support@vortant.com

Telephone: (828) 645-1026 x111

Vortant Technologies, LLC
www.Vortant.com


Step 1. Install the Web Cam

If you haven't installed your webcam yet, that is the first thing to do. Follow the instructions that came with your webcam, and use the installation CD that came with the webcam. When prompted, we recommend you choose "Custom Installation" and "Drivers only", to avoid encountering windows that don't relate to the ViVo Mouse during use.

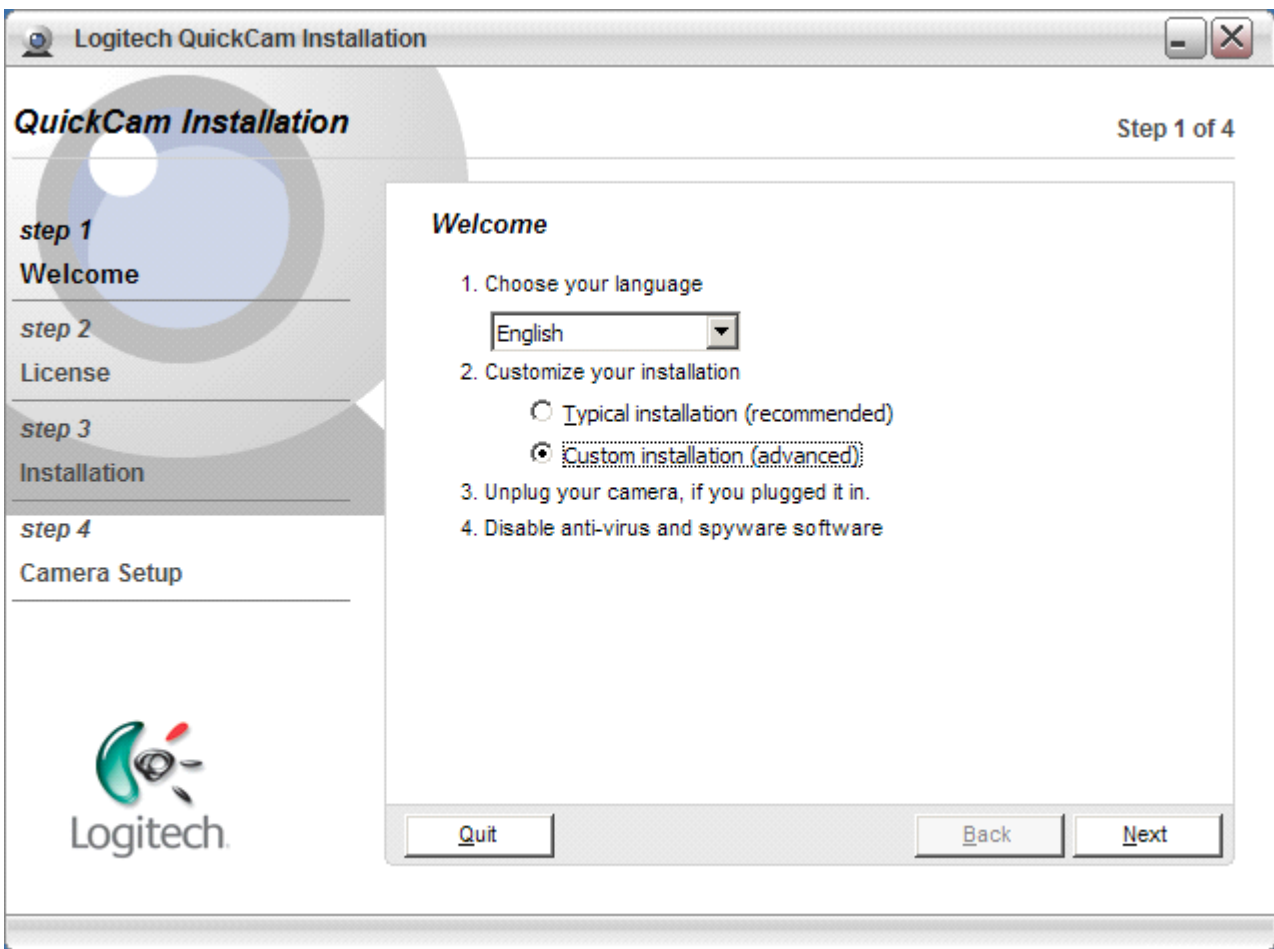

Step 2. Install the ViVo Mouse

Insert the ViVo Mouse CD into your computer, or download if accessing the software digitally. Click on "Setup" or "Setup.exe" to install the program. Or, if you have received the setup files electronically, Click on "Setup" to install the program. Carefully read the Software License Agreement and if you agree, click on "I accept..." and then follow the rest of the instructions.

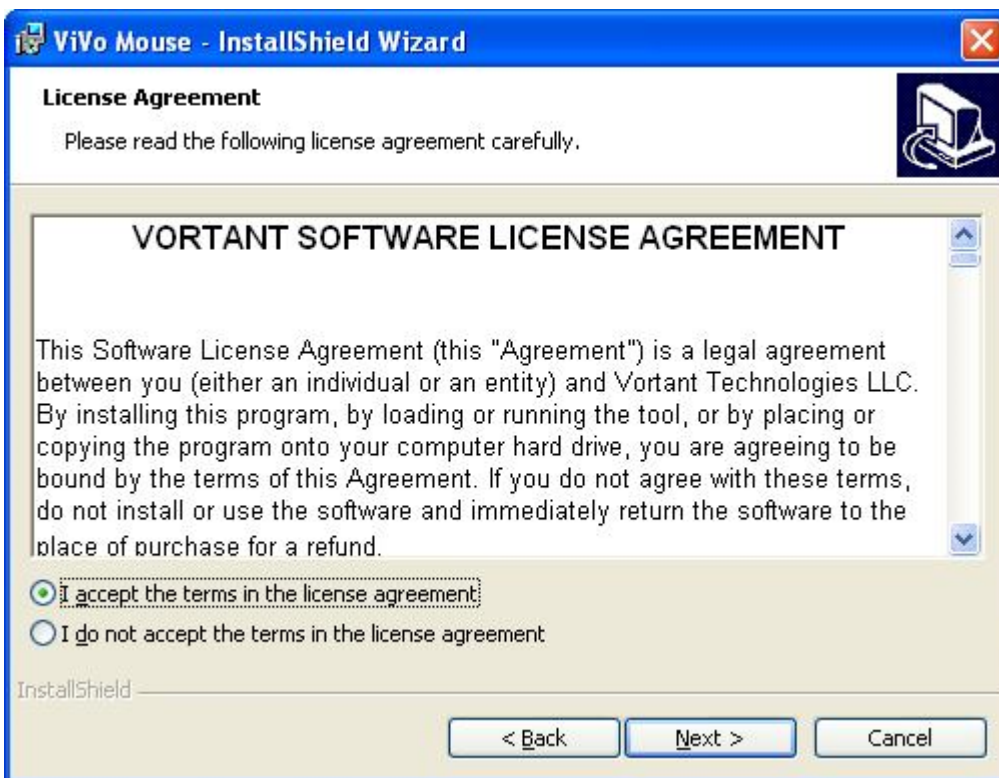

Step 3. Run the ViVo Mouse


After installation, there should be a ViVo Mouse icon on your desktop.

Double click on the icon to start the program. Alternately, click on your computer's **Start** button and find the ViVo Mouse program on your start menu, under the "Vortant" folder.

Windows 7,8 and Vista users:

You may have to right-click on the ViVo Mouse icon and select "Run as Administrator" the first time you start the ViVo Mouse, to avoid a permissions error. If "Run as administrator" does not appear in the right-click menu, Right-click to get the Properties dialog, then under the "Compatibility" tab, check "Run this program as an administrator". Then, after running the ViVo Mouse this way once, you should be able to uncheck that checkbox and it will run normally thereafter.

Note: If the computer asks for your permission to make changes to your computer, with a dialog box similar to the one shown here, say "OK", "Allow", or "Yes".

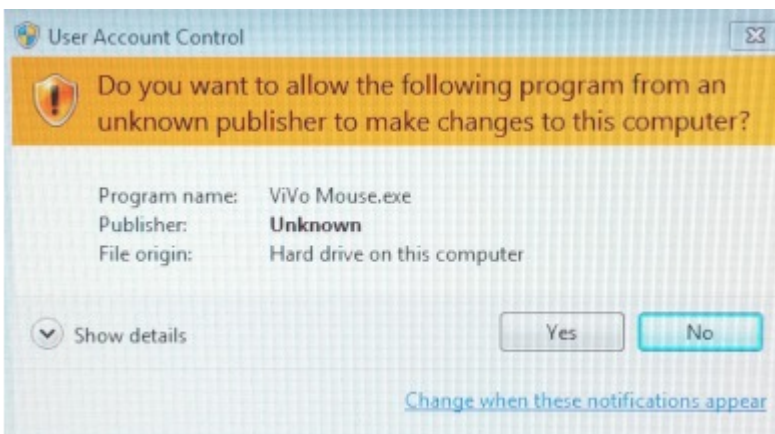

Once you have typed in the activation numbers (found on your CD case), you will see the ViVo Mouse main window:

